

EverFlo



User guide



Required maintenance:

Periodically use a damp cloth to wipe down the exterior case of the device and ensure the holes of the filter door are unobstructed.

How to connect your humidifier?



1. Use the tube on the filter door.








2. Connect the tube to the bottle.



3. Connect the other end of the tube to the oxygen outlet.

Alarms

	The device is turned on and working properly.
Audible Alarm only	Check device is plugged in or there is a power failure.
	Low oxygen condition: continue to use the unit but call your homecare provider.
	Turn the device off, wait 5 min and on again. If condition persists, turn the unit off and call your provider.
	Turn off the unit: check power cord is plugged, check flow meter knob, check nasal cannula, check humidifier connection.
	Turn off the device, connect to a back up oxygen source and call your homecare provider.

If alarm condition persists connect to a back up oxygen and call your homecare provider

This guide is intended to be a quick reference tool for use of the EverFlo. It does not replace the EverFlo User Manual, which provides more complete information and instructions.

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia

+49 7031 463 2254

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585 (toll free, US only)

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)

Philips Respironics International
Headquarters
+33 1 47 28 30 82

Philips Respironics Asia Pacific
+65 6882 5282

Philips Respironics Australia
+61 (2) 9666 4444

Philips Respironics China
+86 800 820 6665
+86 400 820 6665

Philips Respironics Deutschland
+49 8152 93 06 0

Philips Respironics France
+33 2 51 89 36 00

Philips Respironics Italy
+39 039 203 1

Philips Respironics Sweden
+46 8 120 45 900

Philips Respironics Switzerland
+41 6 27 45 17 50

Philips Respironics United
Kingdom
+44 800 1300 845

www.philips.com/respironics

Respironics and EverFlo are trademarks of Respironics, Inc. and its affiliates.

Please visit www.philips.com/respironics



© 2011 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Broudy AE 6/1/2011 MCI 4103861 PN1081279